LUNOS®


DE Lunos® Fluoridgel

EN Lunos® Fluoride Gel

FR Lunos® Gel fluoré

IT Lunos® Gel al fluoruro

ES Lunos® Gel de fluoruro

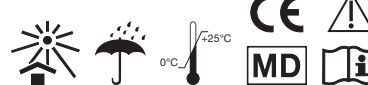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Lunos® Fluoridgel

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EN Operating Instructions

Lunos® Fluoride Gel

Contains sodium fluoride (12300 ppm F-); 10 g Fluoride Gel contains 0.272 g NaF.

Lunos® Fluoride Gel is a medical product. This product is intended only for professional dental application and may be used exclusively by trained specialists! Lunos® Fluoride Gel is free of animal ingredients, gluten, lactose and fructose.

Indication:

- For caries prophylaxis with sodium fluoride
- For remineralisation of the dental enamel
- Reduces increased sensitivity of the teeth and dental necks

Contraindication: The product should not be used in persons in whom control of the swallowing reflex is not assured. Use is contraindicated if lesions or desquamation of the oral mucosa is present. During pregnancy and breast feeding, the products should be used only in accordance with medical advice. To date, adverse side effects related to use of the product have not been observed. Interactions with other products are possible. A risk assessment by a doctor is required in each individual case. Too frequent use or too long contact time or additional fluoride intake exposes the patient to the danger of (dental) fluorosis. The product should not be used in cases of known hypersensitivity or allergies to any of the ingredients.

Warning: Avoid swallowing the gel. Do not use more often than 1 time per week. Consult a doctor in case of overdosage or of swallowing the gel in larger amounts. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state. The product is intended for use on human subjects.

Application: Brush the gel gently onto the surface of the teeth (hazelnut-sized amount) or apply by means of gel carrier (tray) in an amount corresponding to the respective application spoon. Ensure that the gel is left on the teeth for sufficient contact time (at least 2 to 4 minutes). Then briefly rinse the mouth. Wait for up to 60 minutes after gel application before ingesting food.

Storage and shelf life: Store in a dry place. Protect against sunlight. Storage temperatures from 0 °C to 25 °C. The shelf life is 30 months, but after opening is 12 months.

Disposal: Dispose of in accordance with the local official regulations.

Composition: Water, sorbitol, sodium fluoride, disodium hydrogen phosphate, hydrogenated castor oil (PEG-40), hydroxyethylcellulose, phosphoric acid, saccharin sodium, flavouring.

DÜRR DENTAL (Products) UK Ltd. 14 Linnell Way – Telford Way Industrial Estate Kettering, Northants NN16 8PS, United Kingdom ☎ +44 1536526740

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